

Quick Reference Guide

Skytrofa[®] Auto-Injector

For SKYTROFA cartridges

Single-patient use

Watch Video



www.skytrofa.com




This is your Quick Reference Guide

- **Read your Instructions for Use for full instructions before using this Quick Reference Guide.**
- For training video, go to www.skytrofa.com
- If you are unsure about your dose, contact your healthcare provider.



Instructions
for Use

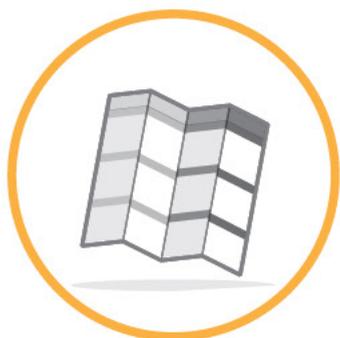

**Quick
Reference Guide**



www.skytrofa.com

Training
Video

Ascendis Pharma Customer Support



1-844-44ASCENDIS

(1-844-442-7236)

Parts overview

Green Top

Starts injection when pressed against skin

Inspection Window

Allows for inspection of mixed medicine

Progress Bar (with six light elements)

Shows progress of mixing and injection

Status Icons

- ✓ **Green check mark**
Injection successfully completed
- 👁️ **Green eye**
Mixed medicine is ready for inspection in window
- 🔄 **Green mixing**
Medicine mixing is in progress

Green Button

To turn on the SKYTROFA Auto-Injector or to reject mixed medicine

Needle

To be screwed onto SKYTROFA Cartridge before inserting the cartridge in green top

Orange Plug

Read more on IFU page 28

SKYTROFA Cartridge

Contains medicine and water for injection. If refrigerated, leave it at room temperature for 15 minutes before use

Battery Icon



Constant green

- ➔ Battery is fully charged
- ➔ The SKYTROFA Auto-Injector is ready to use




Flashing green

- ➔ At least 1 injection remaining, but charging recommended after use



Flashing orange

- ➔ Battery needs charging

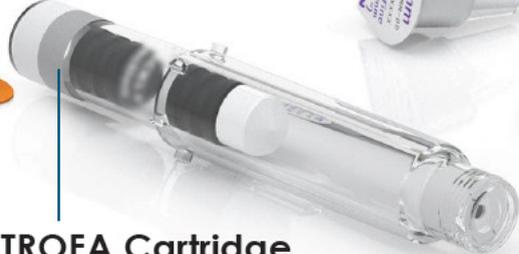

Prepare

1

Check and assemble cartridge and needle

1.1

Check expiration date and cartridge dose on cartridge pack. If refrigerated, allow **15** minutes to reach room temperature.

Expiration date
YYYY - MM
year month

Cartridge dose



1.2

Check expiration date on needle.

Expiration date
YYYY - MM - DD
year month day

Remove paper from needle.



1.3


Screw needle straight and tightly on cartridge. **Do not** remove needle cover.



2

Turn on the auto-injector

Press and release the green button to turn on the auto-injector.

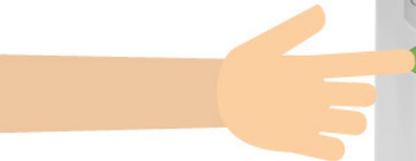

(You will hear **2** loud beeps. Battery icon  will light up and green top will start flashing)

3

Insert cartridge with attached needle

3.1

Needle
Cartridge

Push down cartridge **with attached needle** into flashing green top.



3.2

Click cartridge in place.

(Green top will stop flashing, mixing icon  will light up, and battery icon will switch off)



3.3

After click, remove finger from cartridge.



Mix

4

Wait while mixing

Wait 4 to 8 minutes for the auto-injector to mix medicine.

Watch progress bar gradually light up.




Wait until you hear **2** loud beeps and the entire progress bar flashes.

If your auto-injector turns off automatically read Instructions for Use  IFU page 60.

5

Turn the auto-injector up and down

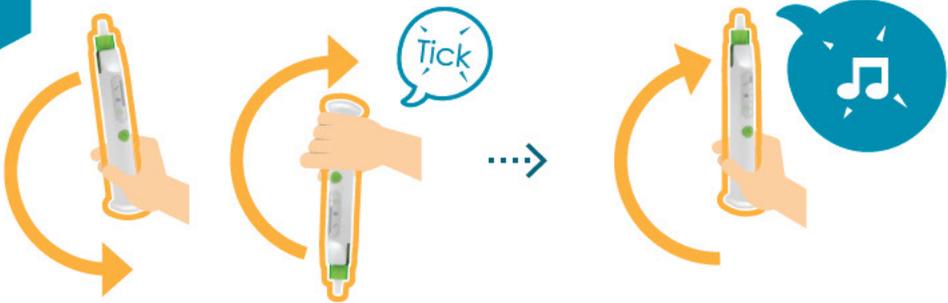

Turn the auto-injector up and down, **listening** for a tick sound each up and down cycle to make sure the turns are correct. **Turn up and down correctly** 5 to 10 times until you hear **2 loud beeps** and the whole progress bar, except the top element, lights up.

Do not press the green button.

If you see a flashing orange mixing icon read  IFU page 58.

6

Finish mixing

Keep the auto-injector upright on a flat surface until you hear **2** loud beeps and the entire progress bar lights up.



6.1

Pull off needle cover.

Do not twist needle cover off.

Keep needle cover for later.

(Eye icon  will light up)



6.2


Inject

7

Check mixed medicine

Medicine is ok to use if it is colorless and clear. It may contain air bubbles. This is normal. Go to Step 8.

Do not use the mixed medicine if it has visible particles. Press the green button for **8** seconds and go to Step 10.

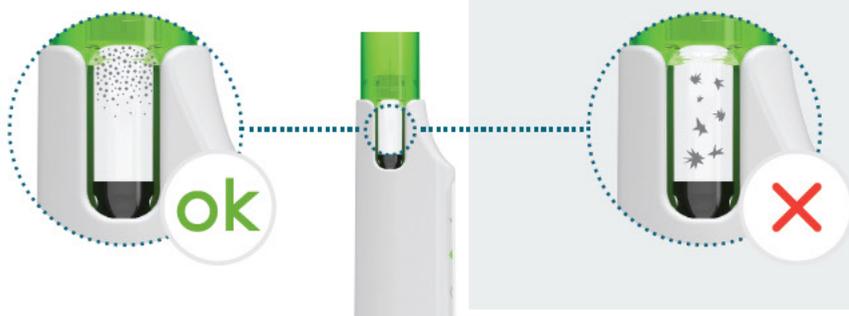


8

Prepare for injection

8.1

Choose an injection site: Stomach (abdomen), thighs, or buttocks. Change the injection site for every injection.

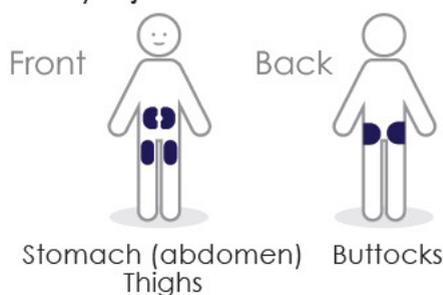


8.2

Make sure your hands are clean using soap and water or hand sanitizer.



8.3

Clean injection site with alcohol wipe. **Do not** inject through clothes.

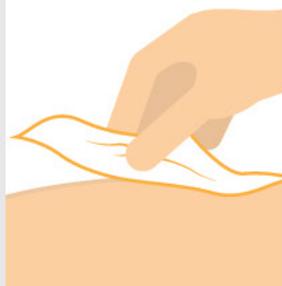

9

Inject medicine

9.1

Press and hold green top against the injection site for **10 to 20** seconds until you hear **2** loud beeps.

(Green top will flash **2** times and check mark icon will light up)



9.2

Remove the auto-injector from skin and wait until you hear **2** loud beeps.

(Green top will start flashing)




After injection

10

Remove cartridge

Press down the needle cover into the flashing green top until you **hear** a click to remove the cartridge.

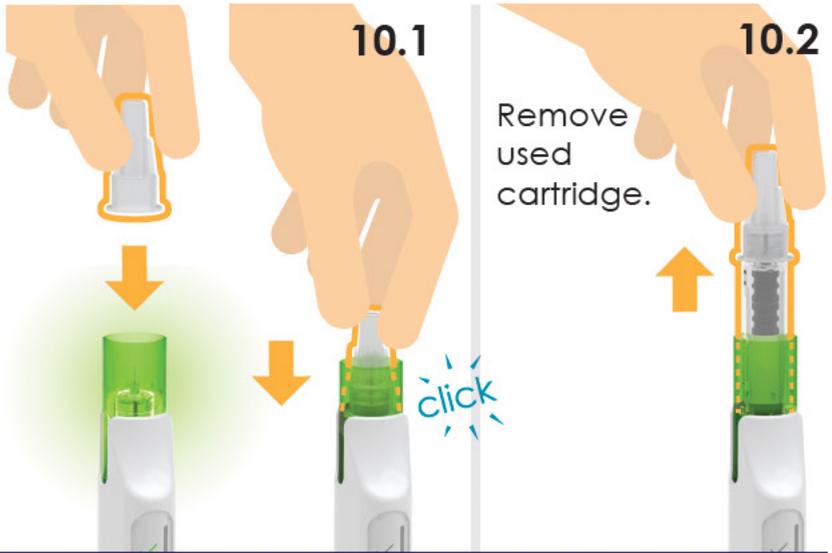

11

Check cartridge and throw away

Check that cartridge is empty.

Put used cartridge and needle in a sharps disposal container.

(More info:
➤ IFU page 46)



Does your weekly dose require 2 cartridges?


Then take the second injection by repeating Step 1 to Step 11 with a new cartridge and needle before continuing to Step 12.

12

Store the auto-injector

Put on the protective cover.

It is recommended to store the auto-injector at room temperature between 59°F to 86°F (15°C to 30°C), to be ready for next use.

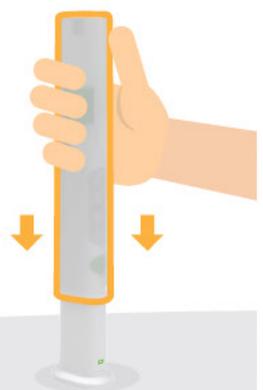


Quick troubleshooting

For all troubleshooting cases and details,
see Instructions for Use (IFU)

➤ IFU page 52

Charging required (Step 2)

➤ IFU page 57

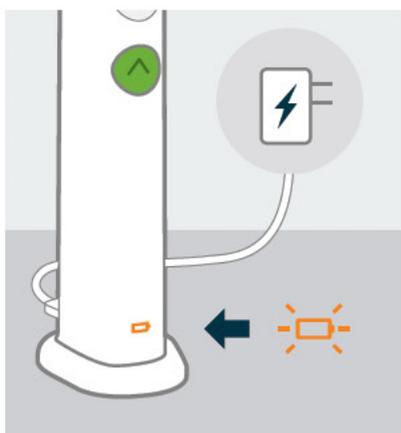


If the battery level
is low:

 You will see a flashing
orange battery icon.

Do this: Insert charging cable
into the SKYTROFA Auto-Injector
at rear lower side and connect
to a power outlet.

SKYTROFA Auto-Injector not upright (Step 4 and 6.1)

➤ IFU page 54

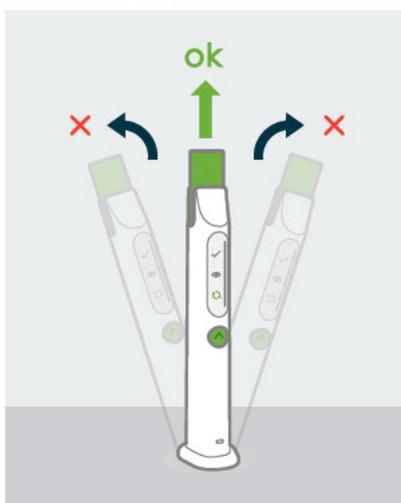


If the SKYTROFA Auto-Injector
is not in an upright position
during mixing and air
removal:

 You will hear a repeating
notification sound. Mixing
icon will flash slowly and
progress bar will freeze.

Do this: Place the SKYTROFA
Auto-Injector in an upright
position and the mixing or
air removal will continue.

Skin contact lost (Step 9)

➤ IFU page 55

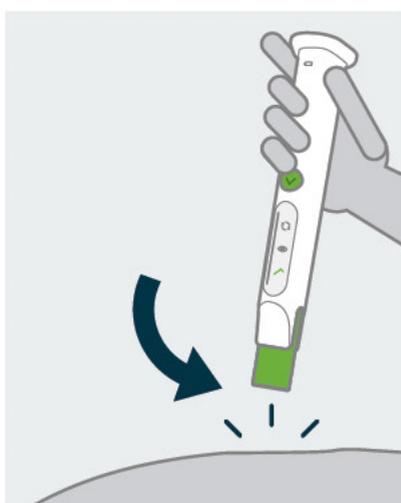

If the green top is removed
from skin before the
injection is complete:

 You will hear a repeating
notification sound. Check
mark icon will flash slowly
and progress bar will freeze.

Do this: Press green top
against skin and the
injection will continue.